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## OPERATING INSTRUCTIONS

## PRELIMINARY REMARK

**Congratulations on purchasing this device!** You will enjoy using your Venen Engel! If you are satisfied with your Venen Engel, we would be delighted if you would recommend it to others.

This manual contains details that users should observe in order to prevent hazards and use the device safely.

If you have any questions about use, accessories, or suggestions, please do not hesitate to contact us!

**Phone:** +49 (0) 666 17 48 91 05  
**Whatsapp:** +49 1573 5990084  
**Mail:** [service@venenengel.de](mailto:service@venenengel.de)  
**Web:** [www.venenengel.de](http://www.venenengel.de)

## BEFORE YOU START

- Please read the operating instructions carefully before using the device.
- Keep this instruction manual in a safe place so that you can refer to it when necessary.
- The Venen Engel must not be used in certain health conditions. Please read the list of contraindications on the last page carefully.

## IMPORTANT SAFETY INSTRUCTIONS AND WARNINGS

**It is important that you read all warnings and precautions in this manual. They are for your safety, to prevent injury, and to avoid damage to the device. injury and prevent damage to the device.**

- Do not use this device if you have a pacemaker, implanted defibrillator, or other implanted metal or electronic device. Doing so may result in electric shock, burns, electrical interference, or death.
- Please read the instructions carefully before using the product.
- Keep this manual near the device so that you can read it during use.
- To avoid hazards, ensure safety, or prevent non-standard operation, the following details are provided details that the user should observe when using the device.
- The device is not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide.
- Mentally impaired persons may only use the device under supervision.
- Repairs, maintenance, and replacement of components must not be carried out during use and may only be performed by specialists authorized by the manufacture. Otherwise, the warranty may be voided.
- If you experience general discomfort or pain during use, please discontinue use immediately.
- When using the device for the first time, the application time should not exceed 5 minutes. For subsequent applications, the duration of use should be gradually extended according to the physical condition of the user. Prolonged use could cause undesirable side effects.
- The product should not be used by infants or small children.
- Keep the device out of the reach of children.
- Do not allow water or other materials (e.g., nails, needles, and other metal objects) to enter the interior of the device.
- If the device does not function correctly, do not open, disassemble, or modify the device under any circumstances, as this may result in fire, electric shock, or other injury.
- Make sure that the plug and your hands are dry when plugging in and unplugging the power plug.
- Only use the device with single-phase 220-230V power. After use, you should unplug the device from the power outlet.
- Make sure to place the device so that you can easily pull out the power plug at any time.

- Make sure that the power cable is not twisted or kinked and do not place any heavy objects (e.g., table and chair legs) on the power cord.
- Only use the device for its intended purpose.
- Do not use the device with multiple sockets. Do not plug the power plug of this device into a socket with other devices.

## SCOPE OF DELIVERY

Control unit

Power plug

Multi-plug air hose set

Leg cuffs

## TECHNICAL DATA

<b>Number of programs</b>	1
<b>Number of chambers</b>	4
<b>Weight</b>	approx. 3 kg
<b>Power output</b>	30 W
<b>Mains frequency</b>	50-60 Hz
<b>Mains voltage</b>	220-240 V
<b>Time setting</b>	up to 30 min
<b>Pressure output</b>	30-240 mmHg
<b>Pump capacity</b>	40 l/min
<b>Fuse</b>	T 0.315 A H
<b>Control unit dimensions</b>	28x28x15 cm
<b>Protection class</b>	II / 
<b>Temperature range</b> (operation & storage)	10-40 °C
<b>Humidity</b> (relative, operation & storage)	30-85

## EXPLANATION OF SYMBOLS USED

Compliant with European standards

Manufacturer's information

Electrical appliances must not be disposed of in household waste!

Follow the instructions for use!

Serial number

Use indoors only

Protection class II: This symbol indicates that the device does not require an electrical ground connection (earth).

Keep away from direct sunlight

Country of manufacture: China

Corrugated cardboard

Polyethylene terephthalate (PET)

Do not wash

Recycle packaging

Disposal instructions France: Dispose of electrical appliances at appropriate locations

Disposal instructions for France: Recycle packaging

## PRODUCT DESCRIPTION

The compression massager consists of a device with a compressor, cuff, and air pressure sensor, which work together as a single unit. The device is connected to the cuffs via a series of tubes. The compression massage is performed by cyclically inflating and deflating the air chambers. The air chambers are filled and deflated one after the other, creating a pleasant, wave-like massage movement.

The pressure in the cuff is controlled by a sensor and microprocessor.

## INTENDED USE

The **Venen Engel Mini** was developed to support your daily well-being.

The sequential inflation of the air chambers creates a **gentle, gliding pressure wave massage** that can convey a **feeling of well-being, lightness, and relaxation**. The **Venen Engel Mini** can also be used as a wellness or beauty treatment to **increase personal well-being** and provide cosmetic care for the skin.

This product is **not a medical device** and is intended solely for cosmetic and relaxation purposes.

## INDICATIONS

---

For relaxing the legs after everyday life, office work, or leisure activities

---

To promote a subjective feeling of lightness and relaxation

---

For pleasant muscle relaxation after prolonged standing or sitting

---

Use as a wellness or beauty treatment to increase personal well-being

---

May improve the appearance of cellulite

---

## CONTRAINDICATIONS

---

Do not use in cases of thrombosis, acute thrombophlebitis, acute heart failure, or arterial insufficiency.

---

Do not use on open wounds, eczema, inflammation, infections, severe sunburn, or rashes.

---

Discontinue use immediately in case of numbness, sensitivity disorders, or pain.

---

Not recommended for use on the legs and abdomen during pregnancy!

---

Do not use on electronic implants (e.g., pacemakers) or metal implants in the treated areas.

---

Do not use in cases of pulmonary embolism or tumors at or near the application site

---

Do not use on children or persons who cannot operate the device safely.

---

## SIDE EFFECTS

---

Redness, feeling of warmth, slight tingling after use.

---

Fatigue or slight dizziness may occur with prolonged use.

---

Pressure points may form if the cuffs are too tight or excessive pressure is applied.

---

Overheating during continuous operation, cable breakage, electric shock if the power supply is damaged.

---

## APPLICATION

### STEP 1: BEFORE USE

#### CONNECT THE AIR HOSES

Connect the hose set to the cuffs, starting with the dark gray connector and then attaching the other gray connectors to their corresponding counterparts.

**Note:** We recommend that you do not disconnect the plug connections afterwards.

**Note:** For users of combination sets with leg cuffs: The two long ends of the hose set belong to the legs and the short end to the abdomen.

#### DEVICE SETUP

Ensure that the device is stable. Also make sure that the device is free-standing and not covered by a blanket or similar. Do not use the device in damp rooms or rooms with high humidity, such as saunas or bathrooms.

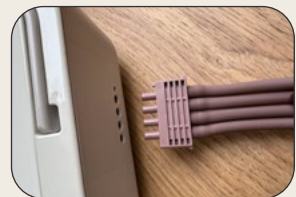
#### PUTTING ON THE LEG CUFFS

To prevent contamination, we recommend wearing leggings or sweatpants during use. Empty your pockets before use! The zippers on the cuffs must be completely closed to prevent the cuffs from opening suddenly and causing possible damage. After you have plugged in the power cord, put on the cuffs, and closed the zipper, the application can be set on the Venen Engel control unit. You will find information on this on the following page.

**Note:** Make sure at all times that the tubes are not kinked or that you are not sitting on them.

#### ATTACHING THE MULTI-PLUG

Insert the multi-plug into the front of the device as far as it will go. Make sure to insert the multi-plug into the device the right way round (small nipple into small hole).

## STEP 2: OPERATING THE CONTROL UNIT

### FOLLOW THE SEQUENCE INDICATED:

1. Select a pressure (30-240 mmHg). Start with a low value.  
You can change the pressure during the massage.
2. Select a duration of up to 30 minutes (maximum 5 minutes for first-time use, then increase gradually).
3. Switch on the device (ON).

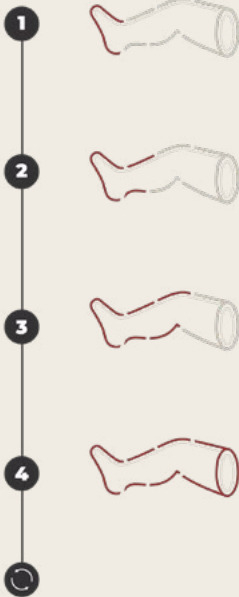
## STEP 3: ENJOY THE MASSAGE

The air chambers in the leg cuffs are inflated one after the other. This creates an optimal sliding pressure wave massage for a relaxing feeling in both legs.

The duration and pressure can also be adjusted during the massage.

### Massage program:

## STEP 4: ENDING AND REMOVAL

Once the set time has elapsed or after pressing the off switch, you can remove the multi-plug from the device. If you do not feel comfortable during the massage, you can also stop the application at any time. You can then open the cuff. To quickly deflate the air cushions after use, we recommend removing the multi-plug and pressing the remaining air out of the cuffs with both hands.

After use, we recommend leaving the gray cuff plugs attached to the cuff and only removing the multi-plug from the device. To store the cuffs, place them on top of each other and roll them up loosely with the hose set attached.

## STEP 5: CLEANING & STORAGE

### **CLEANING**

We recommend using a damp cloth to clean the cuffs and other accessories. For heavy soiling, you can also use a mild fabric cleaner. Before cleaning the cuffs, remove the cuff connectors and allow the cuffs to dry thoroughly. If the cuffs are used by different people, we recommend disinfecting them with a textile disinfectant spray. Use cleaning and disinfecting agents that are free of oil, benzene, alcohol, gasoline, and chemical agents. Do not wash the cuffs in a washing machine or under running water. Switch off the device before cleaning. Disinfectants such as Descosept pur or comparable products with a similar composition are suitable for disinfecting the cuffs.

### **STORAGE**

Store the product at room temperature (minimum 10°C, maximum 40°C) in a well-ventilated, dry, and dust-free room (minimum 30%, maximum 85% relative humidity). Store the product in a place protected from frost, as otherwise damage may occur.

When storing, also take care not to damage the cuffs in particular with sharp objects such as scissors or objects with sharp edges.

For long-term storage, we recommend storing the product in its sales carton.

## **TROUBLESHOOTING**

### **THE DEVICE CANNOT BE SWITCHED ON**

If the device cannot be switched on, please check (A) whether the power plug is correctly inserted into the socket and whether the device is switched on.

### **THE DEVICE PUMPS, BUT ONLY ONE OR NEITHER OF THE TWO CUFFS INFLATE**

If only one or neither of the two cuffs inflates after starting the application, please check (A) whether the multi-plug has been inserted completely and correctly into the device. Also make sure (B) that the air hoses are not kinked, for example because you are sitting on them. Check (C) whether the gray cuff connectors are all correctly connected to the cuff.

## THE CUFFA INFLATE IN THE WRONG ORDER

Ensure that the multi-plug has been inserted into the control unit correctly. The multi-plug is correctly positioned when the small nipples of the multi-plug are located in the designated openings above the air outlets.

## AIR IS FLOWING OUT (AUDIBLE)

- (A) Check the hoses and connectors for damage.
- (B) Check that the multi-plug has been inserted correctly into the device.
- (C) Check whether the hose is kinked or pinched.

## INSTRUCTIONS FOR ACCESSORIES

### ATTACHING THE EXTENSION TAB

## COMBINABILITY OF ACCESSORIES

If you wish to use additional cuffs, please ensure that they have the same number of air chambers as your control unit when purchasing. Only accessories with the correct number of chambers are fully compatible and functional. If the accessories are to be connected to each other, please use the connector provided with the accessories.

## POSITION OF THE ABDOMINAL CUFF (Expected to be available in 2026)

Put on the abdominal cuff so that it connects directly to the leg cuffs. The top chamber of the leg cuff should be flush with the bottom chamber of the abdominal cuff.

Tip: Put on the leg cuffs first. Then lay the abdominal cuff flat behind you on the floor, a couch, or the sofa with the inside facing up. Slowly lie down on your back, pull the cuff to the desired height, and fasten it comfortably around your abdomen. Do not lie on the tubes. The cuff should fit comfortably and not be placed too high on the upper body.

## PUTTING ON THE ARM CUFF (Expected to be available in 2026)

An arm cuff is available for either the right or left arm. When purchasing, be sure to select the correct arm side.

**Note:** If you want to treat both arms, you will need two arm cuffs – one for the right arm and one for the left arm. It is possible to treat the legs and one arm at the same time. However, we recommend treating the arms one after the other and not both arms at the same time. At least one arm should always remain free during use.

## INSTRUCTIONS:

1. Open the clip fastener on the arm cuff if it is still closed.
2. Put the arm cuff on the appropriate arm. Put your hand through the cuff until it is under your armpit.
3. Pull the cuff over your shoulder.
4. Pass the fastener tab behind your back under the opposite armpit and insert the end of the tab into the clip fastener at the front.
5. Adjust the fit by pulling on the end of the fastener tab until the cuff is snug and comfortable.
6. After use, release the arm cuff by pressing the clip fastener on both sides simultaneously.

## SAFETY INSTRUCTIONS

Keep this operating manual in a safe place! Do not dismantle or burn the device. Repairs to the control unit may only be carried out by specialists authorized by Wellcosan GmbH, otherwise the warranty will be void. If the device is damaged, unplug it and switch it off. Also unplug the power cord and switch it off if the device has come into contact with water or other liquids. Do not use the device in damp rooms. Ensure that the device is stable. Do not expose the device to vibrations or throw it. The device must not be covered by other objects during operation. Ensure that the hoses do not become twisted or kinked during use and storage. Keep the device away from heat sources (e.g., direct sunlight, radiators, and others). When not in use, the device must be switched off. The device switches itself off after 10 minutes of inactivity.

When not in use, also unplug the power cord. Always unplug the device from the power outlet before cleaning. Keep the product away from children or persons who have not been instructed in its use.

With the CE marking, the manufacturer or importer confirms that the product complies with the applicable EC directives and meets the „essential requirements“ specified therein.

---

This label on the device means that you must not dispose of it in normal household waste at the end of its service life. Please have it disposed of by an electronic waste disposal company.

---

## WARRANTY

We provide a 24-month warranty on the control unit. In the event of a malfunction, it may be necessary to send the device in for inspection. Please ensure that the shipping box is adequately padded to prevent any damage during transport. Unfortunately, warranty claims cannot be made for defects caused by improper shipping. Warranty conditions: Insofar as there is a legal obligation to provide a warranty, either a replacement will be provided free of charge or the manufacturer will be given the opportunity to repair the device, excluding the right to cancellation or reduction. If, despite several attempts, the repair is still defective or a replacement device is also defective for reasons for which the manufacturer is responsible, the customer has the right to cancellation or reduction.

**Warranty exclusion:** The warranty does not cover defects caused by violent damage, improper operation, external force, or modifications and repairs by third parties, such as defects caused by incorrectly dimensioned or short-circuited fuses, or defects attributable to normal wear and tear.

**Warranty requirement:** The warranty claim is only valid in conjunction with the original proof of purchase (invoice). Therefore, please keep the proof of purchase in a safe place.

If you have any problems or questions when using the system, please do not hesitate to contact us at +49 (0) 666 17 48 91 05 or [service@venenengel.de](mailto:service@venenengel.de).

## DISPOSAL INSTRUCTIONS

Please note that the product contains small amounts of lead. Therefore, please have it disposed of by an electronic waste disposal company. **IMPORTANT: Please do not dispose of the product in your household waste!**

GERMANY:

**Electronic products** must not be disposed of in unsorted municipal waste. Instead, use the available return and collection systems. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products.

Incorrect disposal of electronic products not only wastes resources (most electrical appliances contain valuable raw materials such as gold, copper, silver, etc.), but also endangers the environment due to the sometimes hazardous substances they contain.

ALTERNATIVE: We will be happy to dispose of your old Venen Engel device free of charge. Simply send it to:

**Wellcosan GmbH**  
**Bellinger Tor 16a**  
**36396 Steinau a. d. Str.**  
**Germany**

Please enclose a note requesting disposal.

**Used batteries and accumulators** must be disposed of separately from solid municipal waste. You can do this using the return and recycling systems available to you (e.g., Corepile).

Improper disposal of used batteries and accumulators can have a significant impact on humans and the environment due to the hazardous substances they contain. Recycling preserves valuable raw materials and allows product components to be reused.

FRANCE/SLOVAKIA:

**Electronic products** must not be disposed of in unsorted municipal waste. Instead, use the return and collection systems available. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products.

Incorrect disposal of electronic products not only wastes resources (most electrical

appliances contain valuable raw materials such as gold, copper, silver, etc.), but also endangers the environment due to the sometimes hazardous substances they contain.

**Used batteries and accumulators** must be disposed of separately from solid municipal waste. You can do this using the return and recycling systems available to you (e.g., Corepile).

Improper disposal of used batteries and accumulators can have a significant impact on humans and the environment due to the hazardous substances they contain. Recycling preserves valuable raw materials and allows product components to be reused.

## **IMPORTANT INFORMATION REGARDING ELECTROMAGNETIC COMPATIBILITY (EMC)**

1. This product requires special precautions regarding EMC and must be installed and operated in accordance with the information provided. This device may be affected by portable and mobile RF communication devices.
2. Do not use a mobile phone or other devices that emit electromagnetic fields near the device. This may cause the device to malfunction.
3. Caution! This device has been thoroughly tested and inspected to ensure proper performance and function!
4. Caution: This device should not be used adjacent to or stacked with other devices. If use adjacent to or stacked with other devices is necessary, this device should be observed to verify normal operation in the configuration in which it is used.

## DISCLAIMER

If you have health concerns or are unsure, please consult a healthcare professional before use.

### **NOTE:**

Your Venen Engel product is not a medical device and is not intended for self-treatment. If you are unsure, please consult your doctor.

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© Wellcosan GmbH 2025
Wellcosan GmbH, c/o Caya Postbox 794627,