VENEN ENGEL

KOMPRESSIONS- STRUMPFHOSE

DE	Gebrauchsanweisung
EN	Instructions for use
FR	Mode d'emploi
BG	Инструкции за употреба
HR	Upute za uporabu
EL	Οδηγίες χρήσης
CS	Návod k použití
DA	Brugsanvisning
ET	Kasutusjuhend
FI	Käyttöohje

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THANK YOU FOR CHOOSING A PRODUCT FROM VENEN ENGEL!

Discover the **Venen Engel compression tights**, which bridge the gap between health and modern lifestyle. Thanks to the high-quality material, they are comfortable to wear and can be easily integrated into everyday life without sacrificing comfort. They support blood circulation and are a stylish addition to any outfit. Choose between **tights with a closed toe** or **leggings**, depending on your needs.

The Venen Engel compression tights impress not only with their functionality but also with an appealing design.

Venen Engel compression tights are classified as a **medical device** and should be worn according to the instructions for use. For optimal efficacy and safety, we recommend familiarising yourself with the specific properties and instructions for use.

The **Class II** Venen Engel compression tights have a **pressure of 23-32 mmHg**, and the dynamic effect of **graduated compression** makes them light and easy to wear. When wearing them, circulation is gently stimulated to promote venous return. The compression decreases gradually from the ankle upwards.

The compression tights feature a **reinforced heel**, providing you with the following benefits:

- Perfect fit of the heel in the heel cup for optimal stability of the tights.
- Maximum compression is applied at the ankle to ensure optimal fit and elasticity in the ankle area.

INDICATIONS

- The Venen Engel compression tights are suitable as an aid for the **prevention and supportive treatment of venous system disorders** as well as **lymphoedema and lipoedema**.
- They are particularly **effective against tired and heavy legs** and **reduce evening oedema formation**.
- With constant use, they can prevent or at least delay **pathological venous insufficiency** in predisposed individuals.

- They are suitable for **pregnant women** as well as for **some forms of initial distal insufficiency of the superficial veins** in young or middle-aged people who have to stand a lot or are genetically predisposed.

Please discuss in advance with your doctor whether there are contraindications to treatment with medical compression tights.

ABSOLUTE CONTRAINDICATIONS

- Peripheral arterial occlusive disease in the advanced stage
- Severe pathological oedema of the lower limbs
- Septic phlebitis
- Severe thromboses, such as phlegmasia coerulea dolens
- Lymphangitis
- Peripheral neuropathies in the advanced stage (e.g. due to diabetes mellitus)
- Other disturbances of leg sensitivity
- Intolerance/allergies to the material of the compression tights.

RELATIVE CONTRAINDICATIONS

- Ulcers and open wounds
- Weeping dermatoses
- Immobility
- Stasis dermatitis
- Severe valvular insufficiency of the superficial and deep leg veins
- Deep vein thrombosis or acute thrombophlebitis
- Post-thrombotic syndrome
- In tights with a silicone edge, skin irritations may occur in the event of an allergy to the respective material

SPECIAL NOTES

Medical compression tights should only be worn after consultation with a doctor and sold exclusively by qualified personnel. If pain or skin irritation occurs while wearing the compression tights, remove the tights immediately and consult a doctor or your specialist retailer.

Before use, ensure that the tights do not have any damage or irregularities.

In the event of a serious incident, please inform the manufacturer and the competent authority of the Member State where the user is established.

PRODUCT INFORMATION

Product	Venen Engel medical compression tights
Model	AT-V21 (nude) AT-V22 (black)
Size	20x15x2cm
Weight	≈ 130 g
Material	70 % polyamide, 30 % elastane
Storage instructions	Keep away from heat and direct sunlight.

SYMBOLS USED

Symbol	Meaning
	Manufacturer
	Distributor
	EU standards are met
	Country of manufacture: Italy


Symbol	Meaning
	Recycle packaging
	Hand wash only
	Do not tumble dry
	Do not iron
	Do not dry clean
	Do not bleach
	Recycle packaging and product.
	Packaging: Paper

INSTRUCTIONS AND RECOMMENDATIONS FOR WEARING VENEN ENGEL COMPRESSION TIGHTS

It is recommended to put on the compression tights in the morning after getting up.

The tights should be protected from contact with oils, creams or ointments applied to the legs for medical or cosmetic reasons. Rough fingernails or rings with sharp edges can quickly cause damage to the threads of the product. If the skin is moist, a body powder can help the tights glide more easily over the foot and leg.

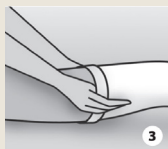1. Roll the tights up to the foot base. It is best to roll the tights until the heel is also turned out.

- Stretch the foot part of the tights with both hands and slip your foot in up to the heel.

- Then roll the tights up along the leg, positioning the heel correctly.
 - Turn the tights back and ensure the correct positioning of the heel.
2. Pull the leg part up to the knee, positioning the tights with the palm of your hand. Never pull directly on the top edge of the tights.
 3. Roll the rest of the tights up to the gusset and also correct the positioning of the tight threads with the palm of your hand, ensuring that the compression is evenly distributed.

MATERIAL COMPOSITION

The composition of the individual products is indicated on the outside of the packaging. The materials from which the products are made are listed below.

EA: Elastane (30%) - **PA:** Polyamide (70%)

CARE INSTRUCTIONS

Venen Engel compression tights can be washed by hand or in the washing machine.

- **Hand washing:** Please use a mild detergent. Rinse thoroughly. Do not wring. Air dry.
- **Machine washing:** Place the tights in a suitable laundry bag or, in the absence of one, in a pillowcase. Do not bleach. Do not tumble dry.

Do not dry clean. Do not iron.

STORAGE INSTRUCTIONS

Liability disclaimer

The manufacturer accepts no liability for damages or injuries in the event of improper or off-label use of the product. If you have any questions or uncertainties, please contact your doctor or specialist retailer.

Manufacturer

Sanyleg Srl S.B. a Socio Unico
Via Albania, 1/3 - 46042 Castel Goffredo
(MN) Italy

Distributor

Wellcosan GmbH, c/o Caya Postbox 794627
Am Börstig 5
96052 Bamberg
Germany

The product meets the requirements of the Medical Device Regulation (EU) 2017/745.

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Ürün, Tıbbi Cihaz Yönetmeliği (AB) 2017/745 gerekliliklerini karşılamaktadır.