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Venen Engel Infrarot Pod Pro

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Venen Engel Infrared Pod Pro

PRELIMINARY REMARK

Congratulations on the purchase of this device!

You will get a lot of enjoyment out of your Venen Engel! If you are satisfied with your Venen Engel, we would be very pleased to receive your recommendation.

These instructions include details that must be observed by the user to prevent hazards and use the device safely.

If you have any questions about use or any suggestions, please do not hesitate to contact us!

Phone: +49 (0) 666 17 48 91 05

Whatsapp: +49 1573 5990084

E-Mail: service@venenengel.de

Web: www.venenengel.de

HelpDesk: help.venenengel.de

BEFORE YOU GET STARTED

- Please read the user manual carefully before using the device.
- Keep this user manual for future reference.
- The Venen Engel must not be used under certain health conditions. Please read the list of contraindications carefully.

1. Product description

The PH-2A is an electrically powered system designed to provide a pleasant warming sensation through integrated far-infrared (8 μm – 14 μm) heating elements.

The device uses evenly distributed heating components within pants and arm cuffs to deliver gentle, adjustable warmth to selected body areas.

The application is intended to support relaxation and general well-being as part of a personal wellness routine.

The system includes a control unit that allows individual adjustment of temperature, time, and heating zones.

Scope of delivery

- (1) Infrared arm cuffs (2x)
- (2) Infrared leg cuff
- (3) Connecting cable cuffs \rightarrow main unit (4x) (already attached to the cuffs)
- (4) Connecting cable main unit \rightarrow outlet
- (5) Main unit

2. Intended Use

The device generates a controlled far-infrared heat that can be perceived as deep and enveloping, delivering a uniform warming effect across the treated body areas.

The application may support relaxation and contribute to a revitalizing body sensation, making it suitable for regular wellness use.

This product is not a medical device and is not intended for the diagnosis, treatment, or prevention of any disease.

Target user

The device is intended for:

- Adults
- Individuals able to understand the instructions

- Users able to perceive and regulate heat independently

Persons with reduced physical, sensory or mental capabilities should only use the device under supervision.

The device is not suitable for:

- Children
- People with impaired temperature or pain sensation
- People who require assistance, are bedridden, or are unconscious
- People who cannot regulate their body temperature on their own
- People with open wounds, acute inflammation, or severely irritated skin in the treatment area
- Pregnant women
- People with implanted electronic devices (e.g., pacemakers)

People with diabetes, cardiovascular diseases, known vascular problems, or impaired temperature perception, as well as pregnant women, should seek medical advice before use.

Operating environment

The device is intended for indoor use only in dry environments.

Recommended environmental conditions:

Storage temperature: -10 - 40°C

Relative humidity: ≤ 80%

3. Improper use

Any use other than the intended use is considered improper.

Examples of foreseeable misuse include:

- Use for weight loss or body shaping purposes
- Use for medical or therapeutic treatments

- Use during sleep
- Use by children
- Using the device outdoors
- Using the device in wet or humid environments
- Operation in folded, bent, or covered condition
- Use on injured, irritated, or inflamed skin

4. Warnings and safety information

General safety information

- Use only as described in this manual
- Do not leave the device unattended during operation
- Do not use while sleeping
- Do not fold or bend the textile components during use
- Do not insert needles or sharp objects
- Do not cover the device during operation

Electrical safety

- Connect only to a properly installed power outlet
- Do not use if the power cable or plug is damaged
- Do not crush or pinch the power cable
- Disconnect from power before cleaning
- Switch off immediately in case of malfunction or unusual heat

Additional warnings/information

- Do not use in bathrooms, showers, or damp environments
- Do not immerse the device in water
- This system adopts dual over-temperature protection: The main controller adjusts temperature gears via NTC temperature sensors which are installed in each zone.

When the actual temperature of a zone exceeds the set value, the main unit stops heating automatically, and the indicator light of that zone turns green.

When the zone temperature drops below the preset value, the indicator turns red to show heating is in progress. The tolerance of temperature is $\pm 5^{\circ}\text{C}$.

As fail-safe protection against NTC sensor or software malfunction, each heating zone is equipped with a mechanical thermostat ($70^{\circ}\text{C}\pm 5^{\circ}\text{C}$). It will cut off the power supply automatically to avoid overheating risks if electronic control fails.

5. Operation / Application

Preparation

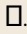
1. Connect the cuffs with the appropriate connector on the main unit.
2. Ensure correct positioning of the connectors (align the colors). The cables are also numbered (see yellow tag) to match the respective connectors.

3. Plug the main unit into a power outlet.
4. Switch on the power switch on the back side of the main unit.

Step-by-step instructions

1. Start the device by pressing the power button .
2. Select the desired zone using the ZONE  button. The indicator light will turn off when a zone is excluded, turn red while heating, and turn green when the set temperature is reached. If the zone fails, the LED will flash.

Zone 1 = right arm, Zone 2 = left arm, Zone 3 = pants frontside, Zone 4 = pants backside

3. Set the temperature (30-60°C) using  +/- . The default temperature is set to 42°C.
4. Set the time (5-95 minutes) using  +/- . The default timer is set to 40 minutes.
5. Start the application with .

Post-Application

- The application stops automatically after the cycle has finished.
- To stop during application, press the  button once more.
- After you have completed the application, first switch off the device , then the power button at the back of the main unit.
- Unplug the main unit.

We recommend a session of 20-40 minutes, 3 times a week at a maximum temperature of 60°C. Start with shorter application times and gradually increase the duration.

The temperature should be set so that the application feels comfortable and is not too hot. The application should never be painful.

We recommend:

- wearing underwear or being naked, i.e. use the device directly on the skin.
- At 50–60°C, limit use to 3 times per week; at temperatures below 50°C, daily use is possible.

- Wipe the inside of the cuffs with a damp cloth after each use and let them dry completely before storing them.

Do not exceed 30 minutes per session and never use the light overnight. Longer sessions do not equal better results—follow recommended usage for safe, effective outcomes.

6. Technical Data

Output

DC 36 V

Power consumption

max. 300 W

Power supply

220-240 V, 50 Hz

Wavelength

8 μm – 14 μm

Protection class

Class II (double insulation)

Protection rating (against wetness)

IPX0 (no protection against water or moisture)

Materials (cuffs)

100% Polyester (PES) bonded with polyvinyl chloride fibres (CLF)

Size

Packaging size: 74x48x45cm

Packaging weight: 1,45kg

Product net weight: 6,78kg

7. Cleaning and Maintenance

Cleaning instructions

- Disconnect from power before cleaning
- Clean textile components with a slightly damp cloth
- Do not use aggressive cleaning agents
- Wipe main unit with a dry cloth only
- Allow all components to dry completely before reuse

8. Storage and Transportation

- Store in a cool, dry place
- Protect from direct sunlight
- Do not bend or compress cables
- Transport in original or protective packaging

9. Troubleshooting

E0: Overload → Reduce the temperature

E1–E4: Sensor error (zones 1–4) → Check that all connections are correct (cuffs & main unit)

E5–E8: Sensor short circuit (zones 1–4) → Turn off the device and contact service.

10. Disposal

Germany

Electronic products must not be disposed of in unsorted municipal waste.

Instead, use the available return and collection systems. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products.

Incorrect disposal of electronic products not only leads to a waste of resources (most electrical appliances contain valuable raw materials such as gold, copper,

silver, etc.), but also endangers the environment due to the sometimes hazardous substances they contain.

ALTERNATIVE: We are happy to take care of the disposal of your old Venen Engel device. Simply send it to:

Wellcosan GmbH

Bellinger Tor 16a

36396 Steinau a. d. Str.

Germany

Please enclose a note requesting disposal.

Other Countries

Electronic products must not be disposed of in unsorted municipal waste. Instead, use the available return and collection systems. This way, we can all contribute to the reuse, recycling, and recovery of electronic products.

Incorrect disposal of electronic products not only wastes resources (most electrical appliances contain valuable raw materials such as gold, copper, silver, etc.), but also endangers the environment due to the sometimes hazardous substances they contain.

11. Symbols

Symbols	Meaning
	Product conforms with relevant EU directives
	WEEE Electrical Product. Do not dispose in household waste!
	Read instructions before use

	Manufacturer
	Made in China
IPX0	Protection rating: device is not protected against wetness or moisture!
	Use only indoors
	Do not wash!
	Do not iron!
	Do not bleach
	Do not dry clean
	Keep away from sharp objects
	Disposal instructions electrical devices France
	Disposal instructions for the primary and secondary packaging France
	Packaging made of corrugated cardboard. Dispose of accordingly.
	Inner packaging made of polyethylen. Dispose of accordingly.
	Serial number

12. Warranty

We offer a 24-month warranty on the control unit. In the event of malfunction, it may be necessary to send the device back for inspection. Please ensure that the shipping box is padded to avoid possible damage during transport.

Unfortunately, no claims under the warranty can be brought for defects caused by improper shipping.

Terms of warranty

Insofar as a legal warranty obligation exists, a replacement will either be provided free of charge or the manufacturer shall have the option to repair the device, under exclusion of the right of cancellation or a reduction in the purchase price. If repeated attempts to effect repair are unsuccessful, or a replacement unit also proves to be defective due to the manufacturer's fault, the customer shall be entitled to cancel the purchase or receive a reduction in the purchase price.

Exclusion of warranty

Excluded from the warranty are defects due to damage caused by the use of force, improper operation, externally applied force, or modifications and repair work performed by third parties, e.g. defects caused by incorrectly rated or short-circuited fuses, or defects attributable to normal wear and tear.

Conditions of warranty

The warranty claim exists only in connection with the original proof of purchase (invoice). Please keep the proof of purchase in a safe place.

Should you have any problems or questions when using the system, please do not hesitate to contact us at +49 (0) 666 174 891 05 or by mail at service@venenengel.de.

13. Liability notice

If you have any health concerns or are unsure, please consult a medical professional before use.

Note::

Your Venen Engel product is not a medical device and is not intended for self-treatment. If you are unsure, please consult your doctor.

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© Wellcosan GmbH 2026
Wellcosan GmbH, c/o Caya Postbox 794627,