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Venen Engel Pro

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Venen Engel Pro

## **PRELIMINARY NOTE**

### **Congratulations on purchasing this device!**

You will get a lot of enjoyment out of your Venen Engel!

If you are satisfied with your Venen Engel, we would be very pleased to receive your recommendation.

This manual contains details that must be observed by the user to prevent hazards and to use the device safely. Should you have any questions regarding its use or any suggestions, please do not hesitate to contact us!

Telefon: +49 (0) 666 17 48 91 05

WhatsApp: +49 1573 5990084

Mail: [service@venenengel.de](mailto:service@venenengel.de)

Web: [www.venenengel.de](http://www.venenengel.de)

## **BEFORE YOU GET STARTED**

- Please read the user manual carefully before using the device.
- Keep this user manual in a safe place so that you can read it again if necessary.
- The Venen Engel must not be used under certain health conditions. Please read the list of contraindications carefully.

## 1. Product Description

The Venen Engel Pro is an electric air compression massage device for your personal beauty and wellness routine at home. The integrated air chambers are rhythmically inflated with air and deflated again, creating a gentle, uniform massage movement.

The Venen Engel Pro is suitable for:

- Beneficial relaxation in everyday life
- A gentle beauty and self-care application
- A pleasant feeling of lightness
- A supple, smooth skin feeling
- A well-groomed and more even-looking skin appearance
- A comfortable wellness massage after long or active days

Various programs, pressure levels, and application times allow for individual adjustment to your personal sense of comfort.

**Note:** The Venen Engel Pro is not a medical device.

### Scope of Delivery

- (1) Control unit
- (2) Power supply plug
- (3) Cuff
- (4) Hose set
- (5) Remote control

## 2. Intended Use

The device was developed to provide a relaxing air compression massage for the legs and to support your daily well-being. It is designed for personal comfort and relaxation after daily activities. The product is not intended for medical purposes, diagnosis, treatment, or prevention of diseases. It is not a medical device.

### Target Users

This device is intended for:

- Adults
- Persons who are capable of understanding the instructions

Persons with reduced physical, sensory, or mental capabilities should only use the device under supervision.

### Health-Related Warnings

Please consult a physician before using the device if you:

- Are pregnant
- Suffer from circulatory disorders

- Have known vascular or heart diseases
- Have recently suffered injuries or undergone surgery, especially in the leg area
- Have reduced sensitivity in your legs
- Use implanted electronic devices such as pacemakers

If you are uncertain about the suitability of the product for your personal situation, please contact a doctor or another medical professional.

### Operating Environment

The device is only intended for indoor use in dry environments.

Recommended environmental conditions:

- Temperature: 10–40 °C
- Relative humidity: 30–85 %

### 3. Improper Use

Any use other than the intended use is considered improper.

Examples of foreseeable misuse include:

- Use for medical treatment or therapy
- Application on injured or irritated skin
- Use while sleeping
- Use in damp environments or near water
- Use while driving or operating machinery
- Alterations or modifications to the device
- Use with components not provided by the manufacturer

The manufacturer is not liable for damages resulting from improper use.

## 4. Warnings and Safety Instructions

### General Safety Instructions

- Read these instructions for use carefully before using the device.
- Keep these instructions for use for future reference.
- Use the device exclusively as described in these instructions for use.
- Do not use the device if it is damaged.
- Keep the device away from children.

Discontinue use of the device if discomfort or unease occurs during application.

### Electrical Safety Instructions

- Use the device exclusively with the supplied power adapter.
- Ensure that the mains voltage matches the device specifications.
- Do not operate the device with wet hands.
- Do not submerge the device in water or other liquids.
- Do not use the device in bathrooms or other damp environments.
- Disconnect the device from the power supply before cleaning.

If the device has come into contact with liquids, disconnect it from the power supply immediately.

### Additional Warnings

- Do not cover the device during operation.
- Ensure that the air hoses are not twisted or kinked.
- Do not place heavy objects on the air hoses.
- Repairs may only be carried out by authorized specialist personnel.

## Residual Risks

Despite careful design and compliance with applicable safety requirements, certain residual risks may remain when using the device. These risks cannot be completely eliminated, but they can be minimized by following the instructions in this manual.

Possible residual risks include:

- Temporary skin redness or pressure marks
- Discomfort due to excessive pressure
- Skin irritation in people with sensitive skin
- Temporary numbness if the cuffs are applied too tightly

To minimize these risks:

- Always start the application with a low pressure level.
- Ensure that the cuffs are positioned correctly and are the right size.
- Stop the application immediately if pain or unusual sensations occur.


## 5. Operation / Application

### Preparation

#### CONNECTING THE HOSE SET AND CUFFS

Attach the connectors of the hoses to the ports on the cuffs. Make sure that the longest hose belongs to the chamber at the foot.


Tip: We recommend not disconnecting the plug connections afterwards.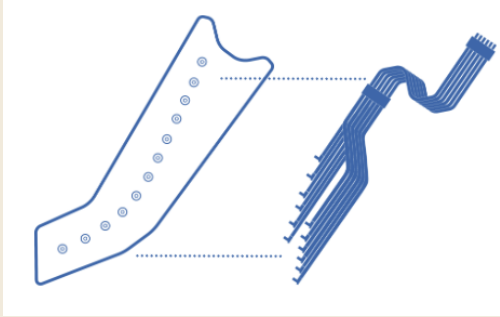

### CONNECTING THE HOSE SET AND CONTROL UNIT

There are two clips on the side of the multi-plug. Press these and then insert the multi-plug into the device.

Tip: Please insert the small nipples of the multi-plug into the designated openings above the air outlets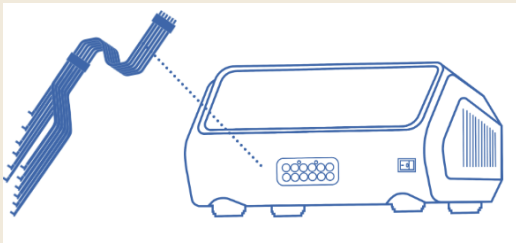


### CONNECTING THE POWER SUPPLY TO THE CONTROL UNIT

Connect the power supply unit to both the device and a power outlet to ensure power supply.


Tip: To protect against contamination, we recommend wearing thin sweatpants or leggings. Ensure your pants pockets are empty. Additionally, close the zippers of the cuffs completely to prevent sudden opening.

### Operation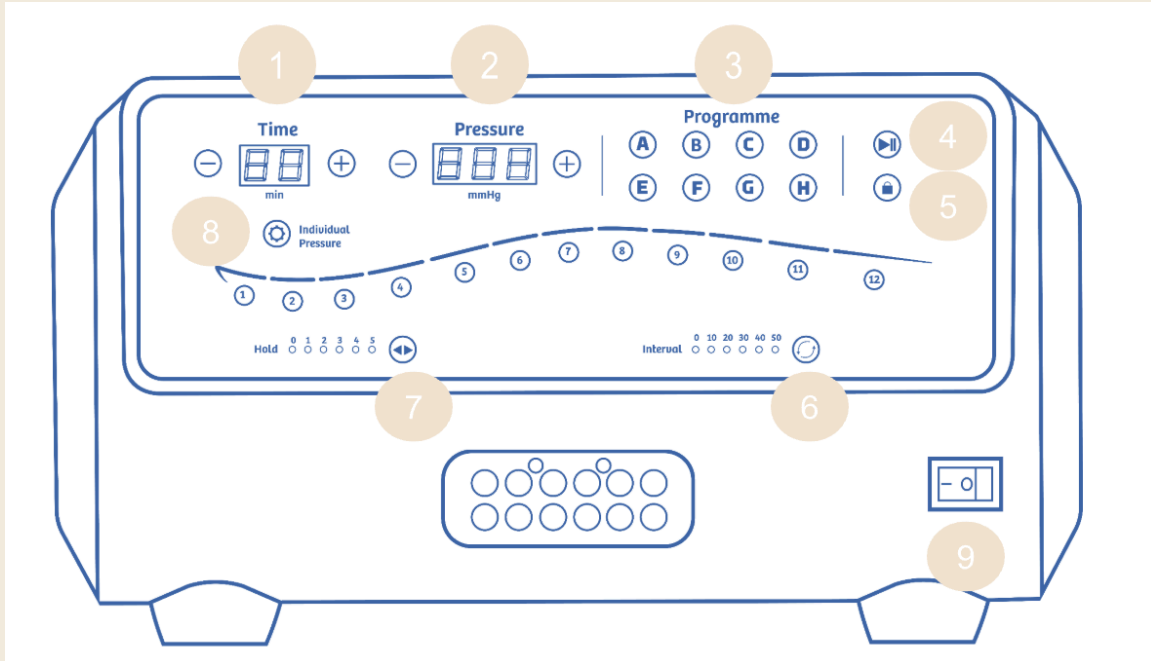


## Control Panel and Remote Control

1. (Remaining) time display: select an application duration between 1 and 90 minutes.
2. Pressure value display
3. Programs A–H
4. Start/Pause
5. Key lock
6. Selection of cycle interval time: (0, 10, 20, 30, 40, or 50 minutes)
7. Selection of compression hold duration: (0–5 seconds)
8. Press this button to select an air chamber and adjust the pressure individually for each chamber.
9. On/Off switch (Power switch)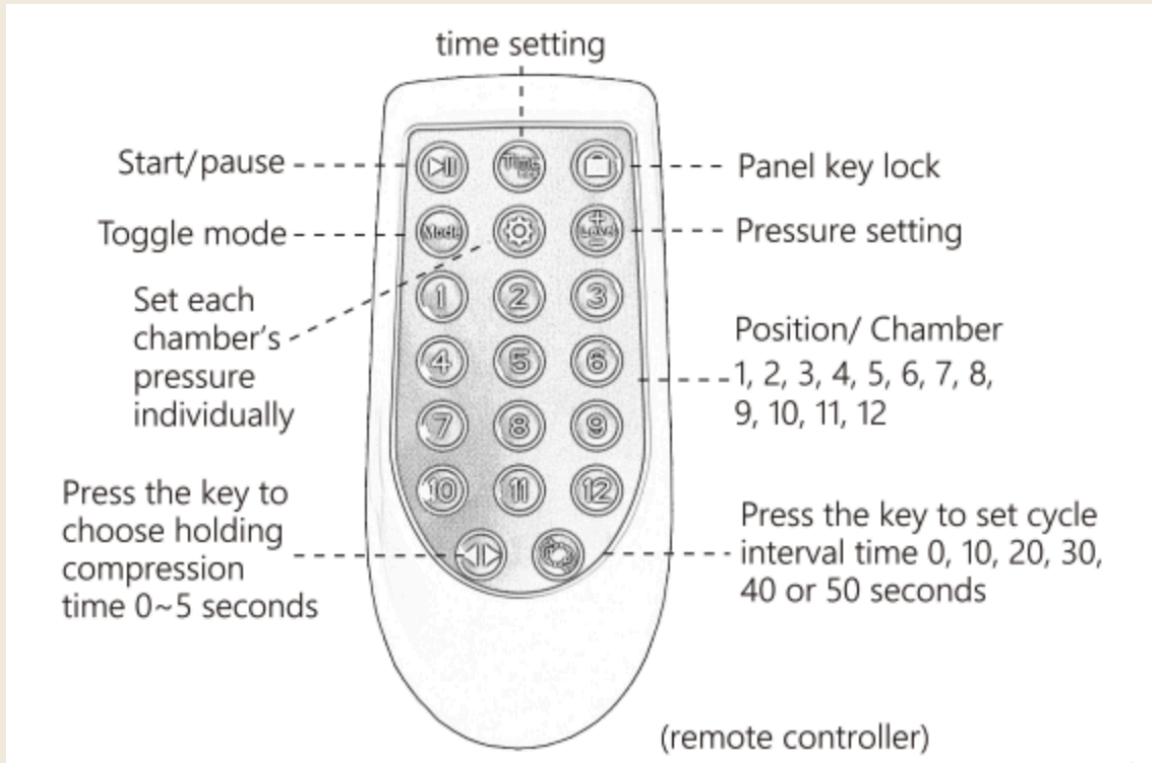

### Step-by-Step Instructions

1. Turn on the device by flipping the switch (9).
2. Select the desired program (3).

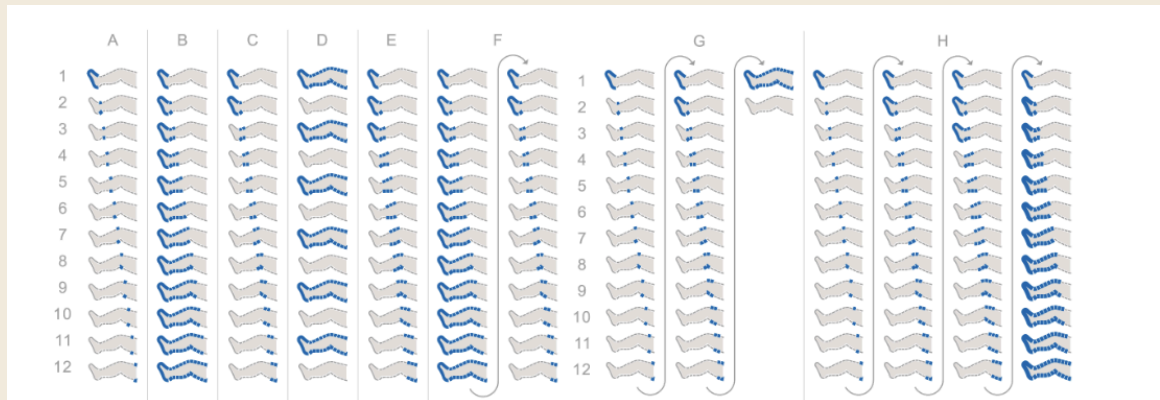3. Set the application time (1).

We recommend an application duration of up to one hour. Always ensure that the application feels pleasant and stop the device immediately if discomfort occurs.

4. Select the pressure level (2).

The device allows a pressure level of up to 240 mmHg. The suitable pressure level depends on personal comfort and the intended cosmetic application. Always start with a low pressure level and increase it gradually if necessary. The application should always feel comfortable and must not cause pain or unusual discomfort.

5. Press Start "" (4) to begin the massage.

The air chambers are automatically inflated and deflated according to the selected program.

## Additional Functions

### Deactivation of Individual Chambers

Individual air chambers can be deactivated by pressing the corresponding chamber button (1–12).

### Individual Pressure Setting

The pressure of each chamber can be adjusted individually. To do this, first select the desired chamber and then set the individual pressure:

1. Set the desired target pressure for the selected chamber.
2. Activate the function by pressing and holding the  button until all number buttons below the chambers flash.
3. Select the desired chamber and adjust the target pressure using "+" or "-".
4. Repeat these steps for all chambers you wish to configure individually.
5. To save the settings, press and hold the  button again.

The flashing symbol indicates that the function has been successfully activated.

### Interval Timer


Allows for pauses between massage cycles.

Set the timer by repeatedly pressing the  button until the desired interval is selected.

The interval timer can be set between 0 and 50 seconds.

### Hold-Timer

Maintains the pressure in the chamber for a defined period of time.

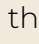Set the hold time by repeatedly pressing the  button until the desired time is selected.

The hold timer can be set between 0 and 5 seconds.

### After Application

1. Turn off the device using the On/Off switch.
2. Disconnect the multi-plug from the main unit.
3. Allow the cuffs to deflate completely before storing them – ideally in the original packaging to ensure optimal protection.

If you wish to accelerate the deflation process, you can gently squeeze the air-filled chambers.

Tip: We recommend leaving the air hose connected to the cuffs, as frequent disconnection can lead to damage to the hoses and connectors.

## 6. Technical Data

<u>Number of programs</u>	<u>8</u>
<u>Number of air chambers</u>	<u>12</u>
<u>Power consumption</u>	<u>78 W</u>
<u>Power supply</u>	<u>AC 220–240 V</u>
<u>Mains frequency</u>	<u>50/60 Hz</u>
<u>Timer settings</u>	<u>5–99 Minutes</u>
<u>Pump capacity</u>	<u>20–25 l/min</u>
<u>Pressure range</u>	<u>30–240 mmHg</u>
<u>Operating temperature</u>	<u>10–40 °C</u>
<u>Humidity range</u>	<u>30–85 %</u>

<u>Dimensions</u>	<u>30 × 25 × 16 cm</u>
<u>Weight</u>	<u>approx.. 5 kg</u>
<u>Protection class</u>	<u>II</u>
<u>Materials</u>	<u>50 % Nylon, 50 % Polyurethane (TPU)</u>

## 7. Cleaning and Maintenance

### Cleaning Instructions

- Disconnect the device from the power supply before cleaning.
- Clean the cuffs with a soft cloth. If necessary, a mild cleaning agent can be used.
- Do not use aggressive or abrasive cleaning agents.
- Allow all components to dry completely before using the device again.
- The main unit may only be cleaned with a dry or slightly damp cloth.

## 8. Storage and Transport

- Store the device in a clean and dry place.
- Avoid extreme temperatures, direct sunlight, and high humidity to preserve the product's lifespan.
- To store the cuffs, lay them flat on the floor and then roll them up loosely. Ensure that the air hoses lie between the cuffs.
- Avoid kinking or twisting the air hoses.

## 9. Disposal

### Germany


Electronic products must not be disposed of in unsorted municipal waste. Instead, use the available return and collection systems. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products. Improper disposal of electronic products not only leads to a waste of resources (most electrical appliances contain valuable raw materials such as gold, copper, silver, etc.), but also endangers the environment due to some hazardous substances contained within them.

### Other Countries

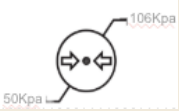Electronic products must not be disposed of in unsorted municipal waste. Instead, use the available return and collection systems. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products. Improper disposal of electronic products not only wastes resources but also endangers the environment due to some hazardous substances contained within them

## 10. Symbols

Symbol	Meaning
	Manufacturer
	Product complies with relevant EU directives
	Made in China
	Read instructions before use

	Serial number
	For indoor use only
	Protect from rain
	Protect from direct sunlight
	Atmospheric pressure range to which the device can be safely exposed
	Humidity range to which the device can be safely exposed
	Storage temperature
	Do not wash
	Outer packaging made of cardboard
	Inner packaging made of polyethylene terephthalate (PET)
	Disposal instructions for electronic products in France
	Disposal instructions for packaging in France

## 11. Warranty

We offer a 24-month warranty on the control unit. In the event of a malfunction, it may be necessary to return the device for inspection. Please ensure that the shipping box is padded to avoid potential damage during transport. Unfortunately, no warranty claims can be made for defects resulting from improper shipping.

### Warranty Conditions

Insofar as a statutory warranty obligation exists, either a replacement is delivered free of charge or the manufacturer has the choice to repair the device, excluding the right to withdraw from the contract or reduce the purchase price. If repeated repair attempts fail or a replacement device also proves to be defective through the fault of the manufacturer, the customer is entitled to withdraw from the purchase or demand a reduction in the purchase price.

### Warranty Exclusion

Excluded from the warranty are defects due to damage caused by the use of force, improper operation, external force, or modifications and repair work carried out by third parties, e.g., defects caused by incorrectly dimensioned or short-circuited fuses or defects due to normal wear and tear.

### Conditions of the Warranty

The warranty claim only exists in conjunction with the original proof of purchase (invoice). Please keep the proof of purchase in a safe place. Should you encounter problems or have questions while using the system, please do not hesitate to contact us at +49 (0) 666 174 891 05 or via email at [service@venenengel.de](mailto:service@venenengel.de).

## 12. Liability Notice

If you have health concerns or are uncertain, please speak to a medical professional before use.

### Note:

Your Venen Engel product is not a medical device and is not intended for self-treatment. If you are uncertain, please ask your doctor.

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© Wellcosan GmbH 2026
Wellcosan GmbH