VENEN ENGEL PLUS

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OPERATING INSTRUCTIONS

PRELIMINARY NOTE

If you have chosen our Leo Edition, please observe the following information on this page.

This special edition differs from our standard version exclusively in selected design and material details:

- **Exclusive Leo** print on the cuffs
- **Controller unit and hoses in a new colour** matching the design
- **Metal logo plate** as a premium design element on the waistband
- **Premium zipper** with slider and end piece made of a zinc alloy for a high-quality look and feel

The functionality, operation and use of your device are identical to the standard version. As usual, you will find all instructions for safe use in the user manual.

Note for people with a zinc allergy:

The metal components of the Leo Edition are made from a zinc alloy. In rare cases, direct skin contact may cause skin reactions in sensitive individuals. If you have a known allergy to zinc or metal alloys, or if you notice skin irritation during use, we recommend discontinuing use.

We hope you enjoy your Leo Limited Edition.

PRELIMINARY REMARK

Congratulations on purchasing this device! You will enjoy using your Venen Engel! If you are satisfied with your Venen Engel, we would be delighted if you would recommend it to others.

This manual contains details that users should observe in order to prevent hazards and use the device safely.

If you have any questions about use, accessories, or suggestions, please do not hesitate to contact us!

Phone: +49 (0) 666 17 48 91 05
Whatsapp: +49 1573 5990084
Mail: service@venenengel.de
Web: www.venenengel.de

BEFORE YOU START

- Please read the operating instructions carefully before using the device.
- Keep this instruction manual in a safe place so that you can refer to it when necessary.
- The Venen Engel must not be used in certain health conditions. Please read the list of contraindications on the last page carefully.

IMPORTANT SAFETY INSTRUCTIONS AND WARNINGS

It is important that you read all warnings and precautions in this manual. They are for your safety, to prevent injury, and to avoid damage to the device. injury and prevent damage to the device.

- Do not use this device if you have a pacemaker, implanted defibrillator, or other implanted metal or electronic device. Doing so may result in electric shock, burns, electrical interference, or death.
- Please read the instructions carefully before using the product.
- Keep this manual near the device so that you can read it during use.
- To avoid hazards, ensure safety, or prevent non-standard operation, the following details are provided details that the user should observe when using the device.
- The device is not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide.
- Mentally impaired persons may only use the device under supervision.
- Repairs, maintenance, and replacement of components must not be carried out during use and may only be performed by specialists authorized by the manufacture. Otherwise, the warranty may be voided.
- If you experience general discomfort or pain during use, please discontinue use immediately.
- When using the device for the first time, the application time should not exceed 5 minutes. For subsequent applications, the duration of use should be gradually extended according to the physical condition of the user. Prolonged use could cause undesirable side effects.
- The product should not be used by infants or small children.
- Keep the device out of the reach of children.
- Do not allow water or other materials (e.g., nails, needles, and other metal objects) to enter the interior of the device.
- If the device does not function correctly, do not open, disassemble, or modify the device under any circumstances, as this may result in fire, electric shock, or other injury.
- Make sure that the plug and your hands are dry when plugging in and unplugging the power plug.
- Only use the device with single-phase 220-230V power. After use, you should unplug the device from the power outlet.
- Make sure to place the device so that you can easily pull out the power plug at any time.

- Make sure that the power cable is not twisted or kinked and do not place any heavy objects (e.g., table and chair legs) on the power cord.
- Only use the device for its intended purpose.
- Do not use the device with multiple sockets. Do not plug the power plug of this device into a socket with other devices.

SCOPE OF DELIVERY

Control unit

Power supply

Multi-plug Hose set

Lymph-Flow Pants

Carry bag & zipper bags

Soft case

TECHNICAL DATA

Number of programs	3 (+4)
Number of compartments	4
Weight	approx. 0.7 kg
Power output	14VA
Mains frequency	50-60 Hz
Mains voltage	110-240V
Time setting	120 min
Printing capacity	30-180 mmHg
Pump capacity	8 l/min

Material (cuffs)	Nylon mit Polyamidfasern
Radio frequency	2,4 GHz
Dimensions of control unit	25x6x7cm
Protection class	II / 
Temperature range (operation & storage)	10-40 °C
Humidity (relative, operation & storage)	30-85

Battery:

Charging time / runtime [in min]	50 / 90
Type	Li-ion
Model number	18600
Voltage	11.1 V
Capacity	1600 mAh
Stored amount of electricity	17.7 Wh

EXPLANATION OF SYMBOLS USED

	Compliant with European standards
	Manufacturer's information
	Electrical appliances must not be disposed of in household waste!
	Follow the instructions for use!
	Serial number
	Use indoors only
	Protection class II: This symbol indicates that the device does not require an electrical ground connection (earth).
	Keep away from direct sunlight
	Country of manufacture: China
	Corrugated cardboard
	Polyethylene terephthalate (PET)
	Do not wash
	Recycle packaging
	Disposal instructions France: Dispose of electrical appliances at appropriate locations
	Disposal instructions for France: Recycle packaging

PRODUCT DESCRIPTION

The compression massager consists of a device with a compressor, cuff, and air pressure sensor, which work together as a single unit. The device is connected to the cuffs via a series of tubes. The compression massage is performed by cyclically inflating and deflating the air chambers. During use, air is pumped into the chambers from the ends of the limbs toward the center of the body.

The pressure in the cuff is controlled by a sensor and microprocessor.

INTENDED USE

The **Venen Engel Plus** is used for air pressure massage of the legs, arms, or waist and promotes a feeling of relaxation, well-being, and lightness. The application promotes a pleasant feeling of lightness in the body and gently stimulates the skin, leaving it looking more vibrant. When used as a wellness or beauty treatment, it can contribute to increased personal well-being and cosmetic skin care.

This product is **not a medical device** and is intended exclusively for cosmetic and relaxation purposes.

INDICATIONS

For relaxation after a long day or after physical activity

To promote a subjective feeling of lighter legs

For pleasant muscle relaxation

May improve the appearance of cellulite

Use as a wellness or beauty treatment to increase personal well-being

Promotes a pleasant feeling of vitality and suppleness of the skin

CONTRAINDICATIONS

Do not use in cases of thrombosis, venous disorders, varicose veins, heart failure, or blood clotting disorders.

Do not use on open wounds, eczema, inflammation, infections, severe sunburn, or rashes.

Discontinue use immediately in case of numbness, sensitivity disorders, or pain.

Not recommended for use on the legs and abdomen during pregnancy!

Do not use on electronic implants (e.g., pacemakers) or metal implants in the treated areas

Do not use in case of fever, infections, acute inflammation

Do not use on children or persons who cannot operate the device safely

SIDE EFFECTS

Redness, feeling of warmth, slight tingling after use.

Fatigue or slight dizziness if used for too long.

Pressure points may form if the cuffs are too tight or excessive pressure is applied.

Overheating during continuous operation, cable breakage, electric shock if the power supply is damaged.

APPLICATION

STEP 1: BEFORE USE

CONNECT THE AIR HOSES

Attach the cuff connectors of the hoses to the connections on the cuffs. Make sure that the longest hose is connected to the foot chamber. Connect the remaining hoses in sequence up to the chamber near the hips. The colors of the connectors and connections help you to determine the connection sequence. We recommend that you do not disconnect the plug connections afterwards.

DEVICE SETUP

Ensure that the device is stable. Also make sure that the device is free-standing and not covered by a blanket or similar. Do not use the device in damp rooms or rooms with high humidity, such as saunas or bathrooms.

PUTTING ON THE CUFFS

To prevent contamination, we recommend wearing leggings or sweatpants during use. Empty your pockets before use! The zippers on the cuffs must be completely closed to prevent the cuffs from opening suddenly and causing possible damage. Ensure at all times that the tubes are not kinked and that you do not sit on them.

CONNECTING THE HOSE SET AND CONTROL UNIT/ATTACHING THE MULTI-PLUG

Insert the multi-plug into the base of the device as far as it will go. Make sure to insert the multi-plug into the device the right way round (small nipple into small hole). There are two clips on the side of the multi-plug. To connect the multi-plug and the device, the clips must be pressed. The multi-plug can then be inserted into the device. The cable must be disconnected in the same way after use.

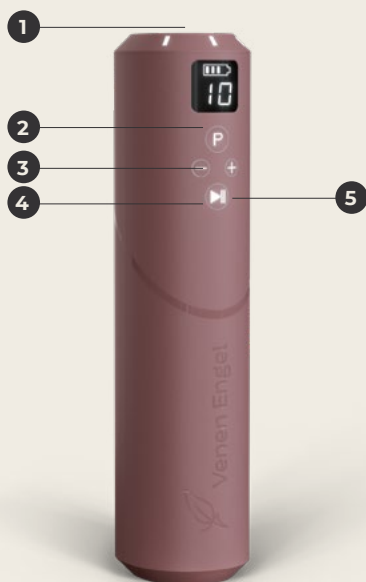
CONNECTING THE POWER SUPPLY TO THE CONTROL UNIT

The device can be operated with or without the power supply, provided the battery is sufficiently charged. Connect the power supply to both the device and a power outlet to charge the battery during use.

STEP 2: OPERATION

1. **TURNING ON THE DEVICE** The on/off switch is located on the top of the device. Press and hold for 3 seconds to turn the device on or off. If the device does not turn on, the battery is probably dead. Use the supplied power adapter to recharge the device.
2. **SELECT PROGRAM** Program A is selected by default. Press the „P“ button to switch between programs A, B, and C.
3. **SELECT PRESSURE** Pressure level 8 is selected by default. You can set the pressure level between 1 and 16 by pressing the „+“ or „-“ buttons. Holding down the corresponding button speeds up the selection of the pressure level.
4. **PLAY / PAUSE** Start your program and print selection by pressing the „Play / Pause“ button. Tip: Pressing the „Play/Pause“ button during a running program activates the pause for 2 minutes. Allow these 2 minutes to elapse without pressing „Play“ again, the device will stop the program and release the air from the chambers.
5. **STOP PROGRAM** To stop a program immediately and release the air, press and hold the „Play/Pause“ button for 3 seconds.

PROGRAMS / GUIDE

With the Venen Engel programs, you can give your body conscious breaks. For lighter legs, better balance, and pleasant relaxation.

AREAS OF APPLICATION

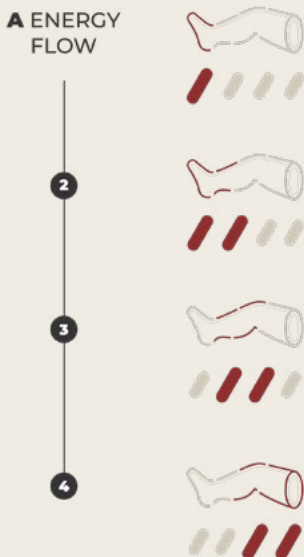
The application can promote skin stimulation and make the skin appear smoother and more even. It can also relieve the feeling of tired and heavy legs and have an overall relaxing effect. Ideal as a wellness or beauty ritual to increase personal well-being. Different areas of focus are set depending on the program.

DURATION AND FREQUENCY

The optimal application time is 30 to 60 minutes – or 10 to 20 minutes for a short break in between. The pressure should be noticeable but comfortable. Gradually find your way to your personal comfort level. The optimal pressure may vary depending on how you feel on a given day. Therefore, adjust it individually as needed. We recommend slowly finding your way to the optimal application pressure. More is not always better: the Venen Engel application should never be uncomfortable or even painful.

Further information on correct use, program selection, and other settings can be found online at: <https://venenengel.de/pages/app>

Procedure Example for program

A ENERGY FLOW

The chambers fill up one after the other, with the pressure in each chamber being maintained until the next one is fully inflated. This creates an even, activating flow that promotes a feeling of well-being in the legs and pleasantly accompanies the body.

B DEEP RELEASE

The chambers fill one after the other and maintain constant pressure throughout the cycle to minimize backflow during the massage. Ideal for relief after a long day or an intense workout. If you are new to this, start with Energy Flow (A) and then slowly move on to Deep Release (B).

C PULSE FLOW

All chambers are inflated and deflated simultaneously, creating a rhythmic pulsating massage with a particularly high number of cycles. The dynamic movement relaxes the body and can make the skin appear more vital and firmer.

IP INITIAL PHASE

Before your selected program/sequence starts, we want to ensure that the leg cuffs are optimally ergonomically adjusted to your legs. To do this, all chambers fill with air simultaneously for 2 minutes. Your program will then start. The process is only active once immediately after starting the control unit and then again after restarting the device.

Please do not interrupt the initial phase by changing the program or switching off the device. Otherwise, the accumulated chamber pressure may build up and become uncomfortable for you.

ESTABLISHING A WIRELESS CONNECTION WITH THE VENEN ENGEL APP

DOWNLOAD THE VENEN ENGEL APP

Download our Venen Engel app from the usual app stores for iOS or Android.

OPEN THE VENEN ENGEL APP

Open the app and select your desired onboarding option.

TURN ON YOUR VENEN ENGEL PLUS CONTROL DEVICE

Pairing will start automatically. Confirm the pairing on your smartphone. If this does not happen, please check the settings on your smartphone.

DONE

You can now operate your control unit via our app and put it aside. Among other things, you can use it to start and adjust the pre-installed programs and sequences, as well as set the pressure and application time.

For more information about the Venen Engel app, visit our website at <https://venenengel.de/pages/app>.

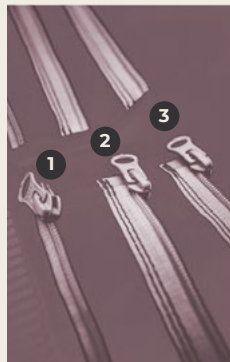
FUNCTIONS THAT CAN ONLY BE CONTROLLED WITH THE APP

DISABLING INDIVIDUAL AIR CHAMBERS

Sensitive areas can be excluded from the massage. All compression chambers are activated by default. If you want to exclude certain chambers from the massage, you can do so here. Simply press the icon for the desired chamber and confirm your selection.

CUSTOMIZATION OF VENEN ENGEL LYMPH-FLOW PANTS

THE THREE ZIPPER POSITIONS

Position 1 = outer zipper = narrowest position = for low leg circumferences

Position 2 = middle zipper = middle position = for medium leg circumferences

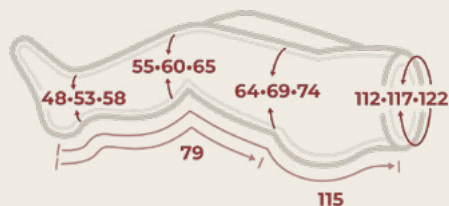
Position 3 = inner zipper = widest position = for large leg circumferences

There are separate zippers for the thighs and lower legs, allowing you to customize the Lymph Flow Pants to your individual needs.

If none of the possible settings fit exactly, you can use our extensions to achieve further size gradations. Just take a look in our **shop**.

To prevent the zipper from opening during use, flip the slider toward your feet after closing the pants. This engages a mechanical lock that prevents the Lymph Flow Pants from opening. After use, you can operate the zipper as normal to take them off.

SIZE OF YOUR VENEN ENGEL LYMPH-FLOW PANTS

Maße (in cm)
Dimensions (in cm)

AFTER USE

TERMINATION AND DISCONNECTION

Once the set time has elapsed or after pressing the off switch, you can remove the multi-plug from the device. If you do not feel comfortable during the massage, you can also stop the application at any time. You can then open the cuff. To quickly deflate the air cushions after use, we recommend removing the multi-plug and pressing the remaining air out of the cuffs with both hands.

After use, we recommend leaving the cuff plugs attached to the cuff and only removing the multi-plug from the device. To store, place the cuffs on top of each other and roll them up loosely with the hose set attached.

CLEANING & STORAGE

CLEANING

We recommend using a damp cloth to clean the cuffs and other accessories. For heavy soiling, you can also use a mild fabric cleaner. Before cleaning the cuffs, remove the cuff connectors and allow the cuffs to dry thoroughly. If the cuffs are used by different people, we recommend disinfecting them with a textile disinfectant spray. Use cleaning agents and disinfectants that are free of oil, benzene, alcohol, gasoline, and chemical agents. Do not wash the cuffs in a washing machine or under running water. Switch off the device before cleaning. Disinfectants such as Descosept pur or comparable products with a similar composition are suitable for disinfecting the cuffs.

STORAGE

Store the product at room temperature (minimum 10°C, maximum 40°C) in a well-ventilated, dry, and dust-free room (minimum 30%, maximum 85% relative humidity). Store the product in a place protected from frost, as otherwise damage may occur.

When storing, also take care not to damage the cuffs in particular with sharp objects such as scissors or objects with sharp edges.

For long-term storage, we recommend storing the product in its sales carton.

TROUBLESHOOTING

THE DEVICE CANNOT BE SWITCHED ON

If the device cannot be switched on, please check (A) whether the device battery is empty and (B) charge the device until it can be switched on.

THE DEVICE PUMPS, BUT ONLY ONE OR NEITHER OF THE TWO CUFFS INFLATE

If only one or neither of the two cuffs inflates after starting the application, please check (A) whether the multi-plug has been inserted completely and correctly into the device. Also make sure (B) that the air hoses are not kinked, for example because you are sitting on them. Check (C) whether the gray cuff connectors have all been correctly connected to the cuff.

THE CUFFS INFLATE IN THE WRONG ORDER

Ensure that the multi-plug has been inserted into the control unit correctly. The multi-plug is correctly positioned when the small nipples of the multi-plug are located in the designated openings above the air outlets.

AIR IS FLOWING OUT (AUDIBLE)

- (A) Check the hoses and connectors for damage.
- (B) Check that the multi-plug has been inserted correctly into the device.
- (C) Check whether the hose is kinked or pinched.

PUTTING ON THE ARM CUFF

(Expected to be available in 2026)

An arm cuff is available for either the right or left arm. When purchasing, be sure to select the correct arm side.

Note: If you want to treat both arms, you will need two arm cuffs – one for the right arm and one for the left arm. It is possible to treat the legs and one arm at the same time. However, we recommend treating the arms one after the other and not both arms at the same time. At least one arm should always remain free during use.

INSTRUCTIONS:

1. Open the clip fastener on the arm cuff if it is still closed.
2. Put the arm cuff on the appropriate arm. Put your hand through the cuff until it is under your armpit.
3. Pull the cuff over your shoulder.
4. Pass the fastener tab behind your back under the opposite armpit and insert the end of the tab into the clip fastener at the front.
5. Adjust the fit by pulling on the end of the fastener tab until the cuff is snug and comfortable.
6. After use, release the arm cuff by pressing the clip fastener on both sides simultaneously.

SAFETY INSTRUCTIONS

Keep this operating manual in a safe place! Do not dismantle or burn the device. Repairs to the control unit may only be carried out by specialists authorized by Wellcosan GmbH, otherwise the warranty will be void. If the device is damaged, unplug it and switch it off. Also unplug the power cord and switch it off if the device has come into contact with water or other liquids. Do not use the device in damp rooms. Ensure that the device is stable. Do not expose the device to vibrations or throw it. The device must not be covered by other objects during operation. Ensure that the hoses do not become twisted or kinked during use and storage. Keep the device away from heat sources (e.g., direct sunlight, radiators, and others). When not in use, the device must be switched off. The device switches itself off after 10 minutes of inactivity.

When not in use, also unplug the power cord. Always unplug the device from the power outlet before cleaning. Keep the product away from children or persons who have not been instructed in its use.

With the CE marking, the manufacturer or importer confirms that the product complies with the applicable EC directives and meets the „essential requirements“ specified therein.

This label on the device means that you must not dispose of it in normal household waste at the end of its service life. Please have it disposed of by an electronic waste disposal company.

WARRANTY

We provide a 24-month warranty on the control unit. In the event of a malfunction, it may be necessary to send the device in for inspection. Please ensure that the shipping box is adequately padded to prevent any damage during transport. Unfortunately, warranty claims cannot be made for defects caused by improper shipping. Warranty conditions: Insofar as there is a legal obligation to provide a warranty, either a replacement will be provided free of charge or the manufacturer will be given the opportunity to repair the device, excluding the right to cancellation or reduction. If, despite several attempts, the repair is still defective or a replacement device is also defective for reasons for which the manufacturer is responsible, the customer has the right to cancellation or reduction.

Warranty exclusion: The warranty does not cover defects caused by violent damage, improper operation, external force, or modifications and repairs by third parties, such as defects caused by incorrectly dimensioned or short-circuited fuses, or defects attributable to normal wear and tear.

Warranty requirement: The warranty claim is only valid in conjunction with the original proof of purchase (invoice). Therefore, please keep the proof of purchase in a safe place.

If you have any problems or questions when using the system, please do not hesitate to contact us at +49 (0) 666 17 48 91 05 or service@venenengel.de.

DISPOSAL INSTRUCTIONS

Please note that the product contains small amounts of lead. Therefore, please have it disposed of by an electronic waste disposal company. **IMPORTANT: Please do not dispose of the product in your household waste!**

GERMANY:

Electronic products must not be disposed of in unsorted municipal waste. Instead, use the available return and collection systems. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products.

Incorrect disposal of electronic products not only wastes resources (most electrical appliances contain valuable raw materials such as gold, copper, silver, etc.), but also endangers the environment due to the sometimes hazardous substances they contain.

ALTERNATIVE: We are happy to take care of the disposal of your old Venen Engel device. Simply send it to:

Wellcosan GmbH
Bellinger Tor 16a
36396 Steinau a. d. Str.
Germany

Please enclose a note requesting disposal.

Used batteries and accumulators must be disposed of separately from solid municipal waste. You can do this using the return and recycling systems available to you (e.g., Corepile).

Improper disposal of used batteries and accumulators can have a significant impact on humans and the environment due to the hazardous substances they contain. Recycling preserves valuable raw materials and allows product components to be reused.

FRANCE/SLOVAKIA:

Electronic products must not be disposed of in unsorted municipal waste. Instead, use the return and collection systems available. In this way, we can all contribute to the reuse, recycling, and recovery of electronic products.

Incorrect disposal of electronic products not only wastes resources (most electrical appliances contain valuable raw materials such as gold, copper, silver, etc.), but also endangers the environment due to the sometimes hazardous substances they contain.

Used batteries and accumulators must be disposed of separately from solid municipal waste. You can do this using the return and recycling systems available to you (e.g., Corepile).

Improper disposal of used batteries and accumulators can have a significant impact on humans and the environment due to the hazardous substances they contain. Recycling preserves valuable raw materials and allows product components to be reused.

IMPORTANT INFORMATION REGARDING ELECTROMAGNETIC COMPATIBILITY (EMC)

1. This product requires special precautions regarding EMC and must be installed and operated in accordance with the EMC information provided. This device may be affected by portable and mobile RF communication devices.
2. Do not use a mobile phone or other devices that emit electromagnetic fields near the device. This may cause the device to malfunction.
3. Caution! This device has been thoroughly tested and inspected to ensure proper performance and function!
4. Caution: This device should not be used adjacent to or stacked with other devices. If use adjacent to or stacked with other devices is necessary, this device should be observed to verify normal operation in the configuration in which it is used.

LIABILITY NOTICE

If you have health concerns or are unsure, please consult a healthcare professional before use.

NOTE:

Your Venen Engel product is **not a medical device** and is not intended for self-treatment.

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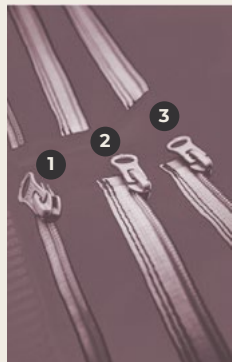
The screenshot displays the 'Gentle Wave (D)' program interface. At the top, the status bar shows the time as 15:16, battery level at 48%, and signal strength. The app header includes a back arrow, the title 'Gentle Wave (D)', and a star icon. The program title 'Gentle Wave (D)' is prominently displayed, followed by the subtitle 'Entspannung & Regeneration'. Below this is a purple 'Starten' button. A section titled '© Dr. med. Dr. med. Dr. med.' introduces the program as a gentle wave for relaxation and stress management, highlighting its benefits for improving sleep, managing stress, and enhancing overall well-being. A 'Programm bearbeiten' button is located below the text. At the bottom, there is a visual representation of the program's effects on the body, showing a person in a relaxed state with a glowing aura around them.

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Wellcosan GmbH, c/o Caya Postbox 794627,
© Wellcosan GmbH 2026